Zero-Resistance™ Bottle Assembly

Zero-Resistance™ Specialty Feeding System Bottle Assembly:

1. To assemble the **Internal Vent System**, connect the reservoir tube to the insert so that they fit snugly together.
2. Place **Internal Vent System** in the bottle.
3. Place the Infant-Paced Feeding Valve (blue one-way valve) into the backside of the nipple/teat.
4. Snap nipple/teat into the collar. Ensure nipple/teat and valve are seated securely within the collar.
5. Affix collar to bottle and turn until closed (do not over-tighten). **If warming the bottle, loosen the collar to prevent fluid loss/leakage.**
6. Do not warm the liquid beyond body temperature of 98.6°F (37°C).

NOTE: Dr. Brown's nipple/teat with the Infant-Paced Feeding Valve (blue one-way valve) in place will fill with fluid as infant elevates tongue to the underside of the nipple/teat during feeding. No manual priming of the nipple/teat is necessary.

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Dr. Brown's® Zero-Resistance™ Bottle Assembly:

1. To assemble the **Internal Vent System**, connect the reservoir tube to the insert so that they fit snugly together.
2. Place **Internal Vent System** in the bottle.
3. Snap nipple/teat into the collar. Ensure nipple/teat is seated securely within the collar.
4. Affix collar to bottle and turn until closed (do not over-tighten). **If warming the bottle, loosen the collar to prevent fluid loss/leakage.**
5. Do not warm the liquid beyond body temperature of 98.6°F (37°C).

NOTE: No air bubbles will be evident during sucking due to Dr. Brown's® Zero-Resistance™ Bottle System.

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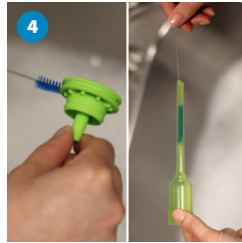
Zero-Resistance™ Bottle Cleaning

Cleaning and Sanitizing

1. Wash cleaning surfaces with solution appropriate for areas with food contact.
2. Prior to initial use, all Dr. Brown's® products should be disassembled and thoroughly cleaned with a dishwashing detergent in water and sterilized/sanitized in a microwave steam bag/electric sterilizer/top-rack dishwasher or steam sterilization (autoclave). Dr. Brown's® products should be washed with hot, soapy water and rinsed well between use.
3. Use Dr. Brown's® Non-Metal Bottle Brush or AccuBrush™ silicone bristles to eliminate debris from the interior of the Dr. Brown's® bottle/vessels and collars.
4. Use the Wire Cleaning Brushes to thoroughly clean the internal vent system (i.e. insert and reservoir tube) and Infant-Paced Feeding Valve (blue one-way valve) only. **Hospital use is recommended to replace the wire cleaning brush every 24 hours after use.**
5. Use the chevron tip at the bottom of either bottle brush to eliminate debris inside the tip of the nipple/teat and assure the nipple/teat hole is clean and open.
6. Prior to each use and after cleaning, pull on the nipple/teat in all directions to make sure it does not tear. **Replace the nipple/teat prior to feeding if the integrity of the nipple/teat is compromised.**
7. The Dr. Brown's® bottle system, nipples/teats, brushes and one-piece silicone pacifiers may be sanitized using Dr. Brown's® Microwave Steam Sanitizing Bags.
8. Disassemble bottle system and place all cleaned parts including bottle/vessel, nipple/teat, collar, internal vent system, brushes and one-piece silicone pacifier in the bag. Place two ounces of water into the bag. **Note: Do not place wire cleaning brush into the Dr. Brown's® Microwave Steam Sanitizing Bag due to metal in the brush.**
9. Each Dr. Brown's® Microwave Steam Sanitizing bag may be used for a total of 20 cleaning cycles following the instructions for use on the bag. **Be aware of the "Warning-Safety Note" to avoid injury to hands/face.**

For questions, concerns, or additional information regarding the assembly, cleaning, and sanitization of Dr. Brown's Medical products, please contact a clinical liaison at medinfo@drbrownsmmedical.com.

***Autoclave (Steam Sterilization) is recommended not to exceed 30 cycles and is NOT recommended for the Infant-Paced Feeding Valve (blue one-way valve), Non-Metal Brush or AccuBrush™.**

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