Zero-Resistance™ Specialty Feeding System:

1. To assemble the **Internal Vent System**, connect the reservoir tube to the insert so that they fit snugly together.
2. Place **Internal Vent System** in the bottle.
3. Place the Infant-Paced Feeding Valve (blue one-way valve) into the backside of the nipple/teat.
4. Snap nipple/teat into the collar. Ensure nipple/teat and valve are seated securely within the collar.
5. Affix collar to bottle and turn until closed (do not over-tighten). **If warming the bottle, loosen the collar to prevent fluid loss/leakage.**
6. Do not warm the liquid beyond body temperature of 98.6°F (37°C).

NOTE: Dr. Brown's® nipple/teat with the Infant-Paced Feeding Valve (blue one-way valve) in place will fill with fluid as infant elevates tongue to the underside of the nipple/teat during feeding. No manual priming of the nipple/teat is necessary.

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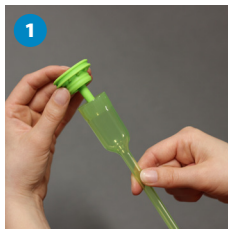
Dr. Brown's® Zero-Resistance™ Bottle Assembly:

1. To assemble the **Internal Vent System**, connect the reservoir tube to the insert so that they fit snugly together.
2. Place **Internal Vent System** in the bottle.
3. Snap nipple/teat into the collar. Ensure nipple/teat are seated securely within the collar.
4. Affix collar to bottle and turn until closed (do not over-tighten). **If warming the bottle, loosen the collar to prevent fluid loss/leakage.**
5. Do not warm the liquid beyond body temperature of 98.6°F (37°C).

NOTE: No air bubbles will be evident during sucking due to Dr. Brown's® Zero-Resistance™ Bottle System.

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