



## Instructions for Using the Dr. Brown's® Microwave Steam Sanitizing Bags

1. Clean and rinse all items as recommended in the Dr. Brown's Medical Manufacturer Suggested Guidelines for Hospital and Clinical Environments.
2. Place clean and rinsed product parts in the Microwave Steam Sanitizing bag. Make sure all parts are disassembled. Do not overfill steam bag with parts.
3. Add 2oz/60 mL of water to the microwave steam bag.  
**IMPORTANT NOTE** – Using less than the appropriate amount of water could distort the shape of the products.
4. Seal bag tightly.
5. Put bag with products inside, into microwave oven on center of turntable.
6. Microwave on High Power per the time chart below:

1.5 Minutes for 1100 W+  
3 Minutes for 800 - 1100 W  
5 Minutes for 500 - 750 W

7. Using an oven mitt or potholder, carefully remove the bag from the microwave.  
**WARNING: Be careful as steam and water will be very hot. Avoid touching the Drain Spout (indicated in RED) on the microwave steam bag.**
8. Cool 10 minutes.
9. Use Drain Spout to pour out excess water before removing the sanitized parts.  
**NOTE: Each Dr. Brown's Microwave Steam Bag can be used for a total of 30 cycles.**

### **WARNING – SAFETY NOTE**

- Be careful when removing bag from microwave as water and steam will be very hot.
- Only sanitize parts deemed safe for microwaves.
- Steam bag is intended only for plastics that are microwave safe.
- Do not place the steam bag in a conventional oven.
- Do not place the steam bag on the metallic grill in some microwaves.
- Be sure to follow the time chart as heating longer than the recommended time may distort shape of parts.