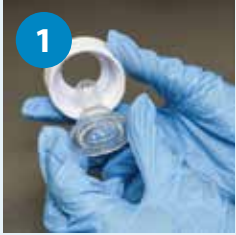
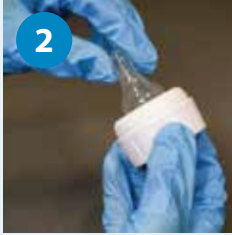
Zero-Resistance™ Bottle | Biberon Zero-Resistance

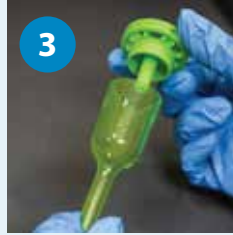
Biberón Zero-Resistance™


1


2


3

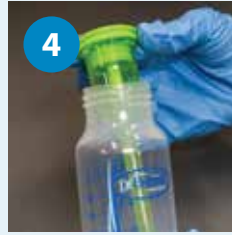4

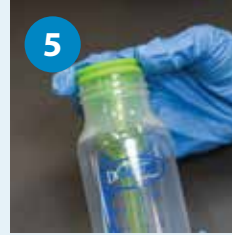5

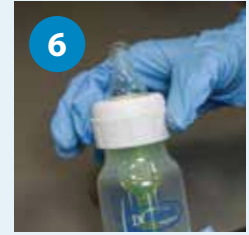6

1. Insert the nipple into the collar.
2. Make sure the nipple is fully seated in the collar.
3. Assemble the Internal Vent System by placing the insert into the reservoir to snugly fit together.
4. Place the assembled Internal Vent System into the bottle.
5. Ensure the insert makes full contact with the top of the bottle.
6. Affix collar to bottle and turn until closed (do not over-tighten).

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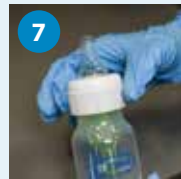
6

7

Specialty Feeding System includes the Infant-Paced Feeding Valve (blue one-way valve) creating a "compression" nipple.

1. Insert the nipple into the collar.
2. Insert the Infant-Paced Feeding Valve (blue one-way valve) into the assembled nipple and collar.
3. Make sure the nipple, collar, and valve are fully seated.
4. Assemble the Internal Vent System by placing the insert into the reservoir to snugly fit together.
5. Place the assembled Internal Vent System into the bottle.
6. Ensure the insert makes full contact with the top of the bottle.
7. Affix collar to bottle and turn until closed (do not over-tighten).

Note: With the Infant-Paced Feeding Valve (blue one-way valve) in place, the nipple will fill with fluid as infant elevates tongue to underside of the nipple during feeding. No manual priming of the nipple is necessary.

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Cleaning Instructions

CAUTION: Do not use the wire cleaning brush to clean the nipple as this will cause it to tear.

1. Use clean hands (consider wearing protective gloves) and clean surfaces when handling.
2. Before first use and after each use, disassemble and wash all parts with soap and water.
3. Consider using one of Dr. Brown's nonmetal brushes, Non-Metal Bottle Brush or AccuBrush™, to remove debris from bottle and nipple parts.
4. Use the chevron tip at the bottom of either nonmetal brush to safely clean nipple.
5. Use a Dr. Brown's Wire Cleaning Brush to thoroughly clean the Internal Vent System (i.e., insert and reservoir).
6. Rinse well and air dry.
7. Prior to each use, pull on nipple in all directions to make sure there are no tears. Be sure the nipple hole is clear.
Replace the nipple if the integrity is compromised.

Sanitization

WARNING: Do not microwave wire cleaning brush.

8. After washing, consider sanitizing before first use and every 24 hours thereafter. Use the Dr. Brown's Microwave Steam Sanitizing Bag (or comparable process). **Follow instructions printed on the bag and observe the "WARNING-SAFETY NOTE" to avoid injury.**

Dr. Brown's bottle systems, nipples, and Infant-Paced Feeding Valve (blue one-way valve) have been tested for functionality up to 30 cycles of steam sterilization (autoclaving). Letter to support Steam Sterilization – Autoclave available upon request.

For questions or concerns, or additional information regarding the assembly, cleaning, and sanitization of Dr. Brown's products, please email: medinfo@drbrownsmc.com.

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All Rights Reserved. Tous droits réservés.